

READYWRAP™

MANUFACTURED BY SOLARIS

Easy Alternative

to Elastic
Compression

About ReadyWrap



Donning is simple and easy with color-coded hook and loop closures.



Easily adjust for daily girth fluctuations and personal comfort.



Prevent gapping with overlapping straps and garments.



Low-stretch material helps provide the compression you need when you need it.



*Choose
between*

Black

Beige

SOLARIS
FREEDOM TO LIVE

T: (855) 892-4140
F: (414) 892-4150

info@solarismed.com
solarismed.com

 /solarisinc

 /solarisinc

 /solarisinc

How to Don the ReadyWrap Calf



A clean liner should always be donned before applying lower extremity ReadyWrap garments.

- 1 Place the spine of the garment against your calf and fasten the shortest strap with white hook just above your ankle bone.



- 2 Fasten the remaining straps in order from light blue hook to dark blue, small piece of hook to big, adjusting for comfort as needed.

For more information about ReadyWrap, visit solarismed.com.

Easy to Care For

Washing



- 1 Hand wash only.
- 2 Use warm water with a drop of dish soap or mild detergent.
- 3 Do not use bleach, fabric softeners, or other additives.

Drying



- 1 Lay flat to dry.
- 2 Do not use a drying machine or an iron.
- 3 Drying time can be shortened by laying the garment on a towel and rolling it inside the towel to collect excess water.

Warranty

ReadyWrap garments are protected against manufacturer defect by a six month warranty.

Find a Retailer at:
www.solarismed.com/where-to-buy