

TRIBUTE *Night*™

MANUFACTURED BY SOLARIS

Eliminate Nightly Wrapping and Maintain Gains Made in Therapy



About TributeNight



Save time and reduce stress by eliminating nightly wrapping¹.



Individually designed to provide the support you need while you sleep.



Machine wash and dry garment is easy to care for and maintain.



Backed by an industry leading One Year Warranty and Solaris' exclusive SureFit GuaranteeSM.

Choose
between



¹Falk, M., McCusker, F., Solaris, (2011).
Nightly Lymphedema Management: A Comparison Case Study.

SOLARIS
FREEDOM TO LIVE

T: (855) 892-4140
F: (414) 892-4150

info@solarismed.com
solarismed.com

 /solarisinc

 /solarisinc

 /solarisinc

How to Don the TributeNight Arm



- 1 Fold the upper half of your TributeNight over the lower half. Pull the garment over your hand with the sewn seam aligning with your thumb.



- 2 Roll the upper half of the garment onto your arm, twisting the garment slightly toward your body to align the sewn seam to the center of your armpit.

For more information about TributeNight, visit solarismed.com.

Easy to Care For

Washing



- 1 Machine wash with mild detergent on permanent press.
- 2 Double rinse to remove soap residue.
- 3 Do not use fabric softener, bleach, or other additives.

Drying



- 1 Tumble dry on permanent press.
- 2 Drying time can be shortened by laying the garment on a towel and rolling it inside the towel to collect excess water.

Warranty

TributeNight garments are protected against manufacturer defect by a One Year Warranty and Solaris' exclusive SureFit GuaranteeSM.

Find a Retailer at:
www.solarismed.com/where-to-buy